


Interpretation of Results

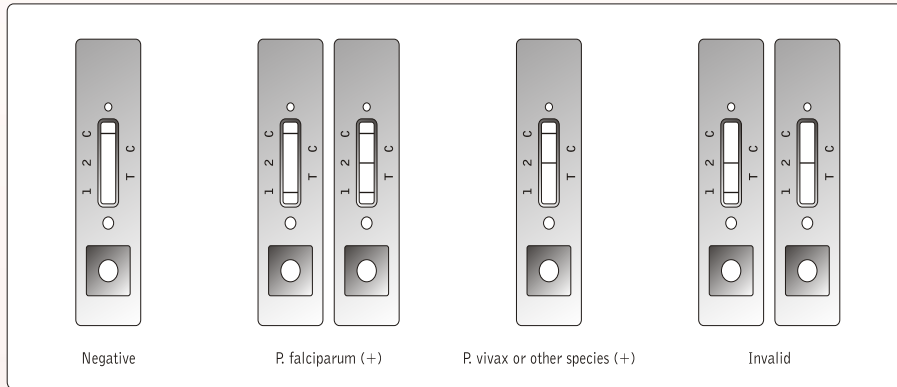

Ordering Information

PRODUCT CODE	PRODUCT DESCRIPTION	PACK SIZE
FRDEM005	ErbaQik Malaria Pf/Pan (Ag)	20Tests

For further information, please contact :

ERBA diagnostics Mannheim GmbH

Mallaustrasse 69-73, D-68219 Mannheim / Germany

Telephone: +49 621 8799770 Fax: +49 621 8799688


Website: www.erbamannheim.com Email : sales@erbamannheim.com



TM

ErbaQik

Malaria Pf/Pan (Ag) Device

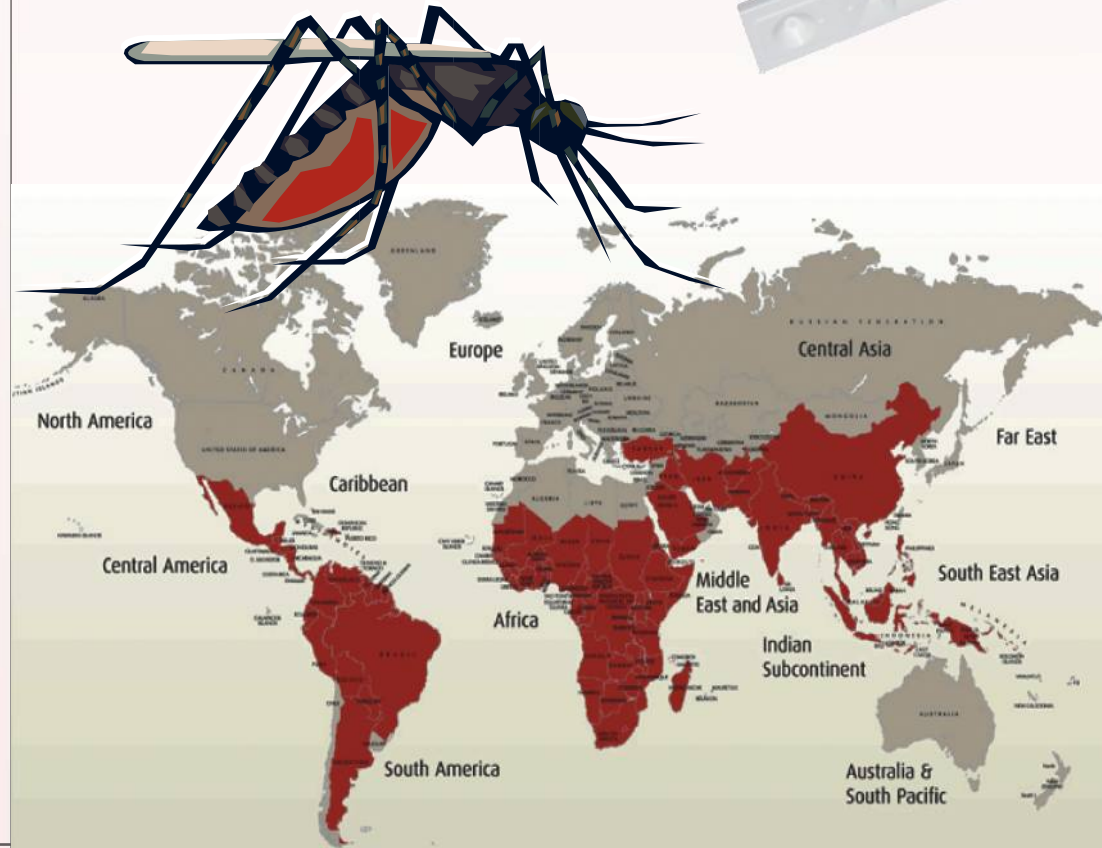



Rapid Screening made easy

A Qik screening test device for detection of Malaria Pf/Pan (Ag)...




- Qualitative detection of Malaria (Ag) in human whole blood.
- Differential detection of *P.falciparum* and other species (*P.vivax*, *P.ovale* and *P.malariae*).
- Convenient pack size of 20 tests.
- Room temperature storage.


Test Procedure:

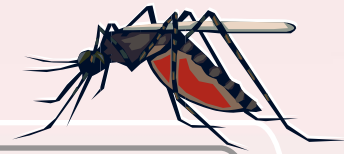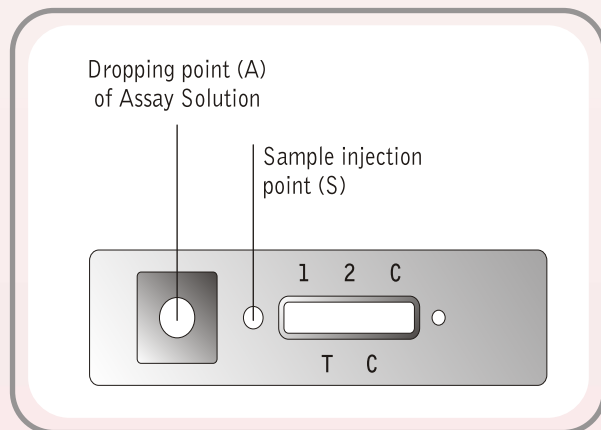


Remove device
from pouch

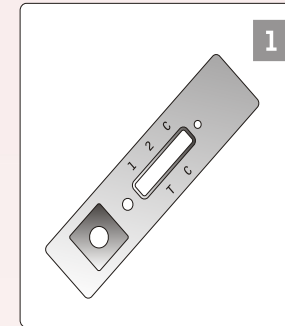


Prick finger or
collect from vein

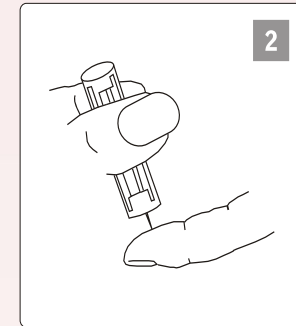


Draw 10ul sample

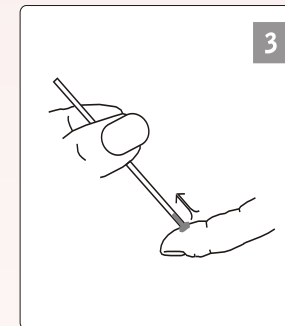

Dispense sample
in the sample well

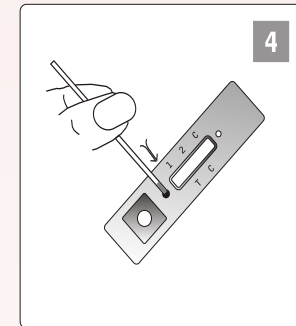

Add 4 drops of
Assay soln. in
assay well.

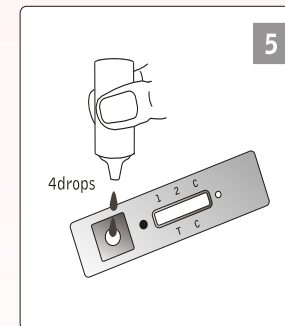

Read result in
15-20 minutes

