

## The first line of defence in your diagnosis

Urine Reagent Strips for accurate and reliable results...

### Uro-dip 2VGK ....

Visual check strip

Measures Urinary Glucose and Ketones

Provides indication for diabetic ketoacidosis

Analysis time of 60 secs

Convenient pack of 50 strips



### Uro-dip 2VPG ....

Visual check strip

Measures Urinary Albumin and Glucose

Analysis time of 30 & 60 secs

Convenient pack of 100 strips

## Analyzing a urine sample...

Dip test strip in fresh urine sample for 1 second making sure the reagent pads are wet

Remove test strip from container, run the edge against rim to remove excess urine

Compare the colour of reagent pads with corresponding colour chart on the bottle



Specifications are subject to change without prior notice.

### **ERBA diagnostics Mannheim GmbH**

Mallastrasse 69-73, D-68219 Mannheim / Germany

Tel.: (+49) 0621 8799770 • Fax: (+49) 0621 8799688

Email: [sales@erbamannheim.com](mailto:sales@erbamannheim.com) Website: [www.erbamannheim.com](http://www.erbamannheim.com)